



DIRECTIONS FOR USE

Polishing Acrylic Based Denture Material:

1. Mix product with water to make a thin paste (about 2 parts Acrylux™ to 1 part water)
2. Place paste on the surface of the appliance to be polished and apply even pressure against dental lathe ragwheel.
3. Check polished surface instantly, as Acrylux™ polishes quickly. If more polishing is required, check appliance frequently to avoid over polishing.

Polishing Nylon Based Denture Material:

1. Follow directions 1 and 2 above. The appliance will require greater pressure and time, but continue to check the surface frequently to examine for acceptable smoothness. Be sure to use plenty of water and keep the appliance surface wet.

For more information regarding Acrylux™, please contact us directly at:

888-355-8548